


FIELD CONNECTION WIRES-----18 GAGE
 POINT WIRES-----22 GAGE
 STATION WIRES-----22 GAGE
 AUDIO WIRES-----22 GAGE SHIELDED

IT IS RECOMMENDED THAT A FIELD CONNECTION CABLE RUN FROM THE CONTROL PANEL BE LOOPED DOME LIGHT JUNCTION BOX TO DOME LIGHT JUNCTION BOX FOR EACH SIDE OF A HALL WAY.


NOTE!
 RUNS OVER 1000FT INCREASE FIELD CONNECTION WIRES TO 16 GAGE

A CONNECTOR IS SUPPLIED WITH EACH PATIENT STATION. WIRES NOT USED MUST BE ELECTRICALLY ISOLATED

POINT (1-80)
 900-203 CONTROL

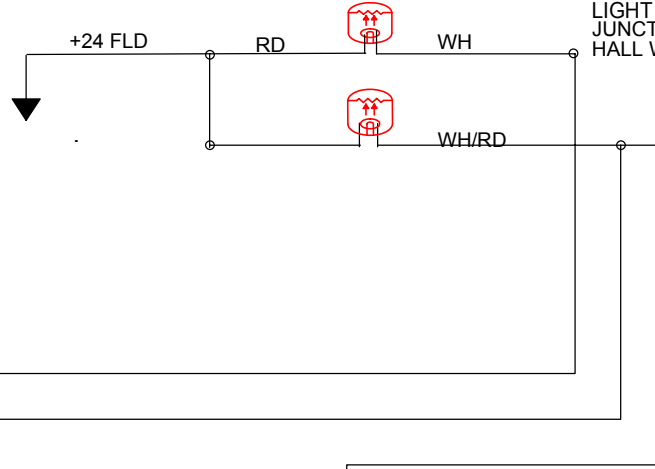

SHIELD
 AUDIO
 AUDIO
 ALL CALL
 EMERGENCY MID
 EMERGENCY LOW
 CHASSIS GROUND
 GROUND
 ROOM COMMON
 + 24 FLD

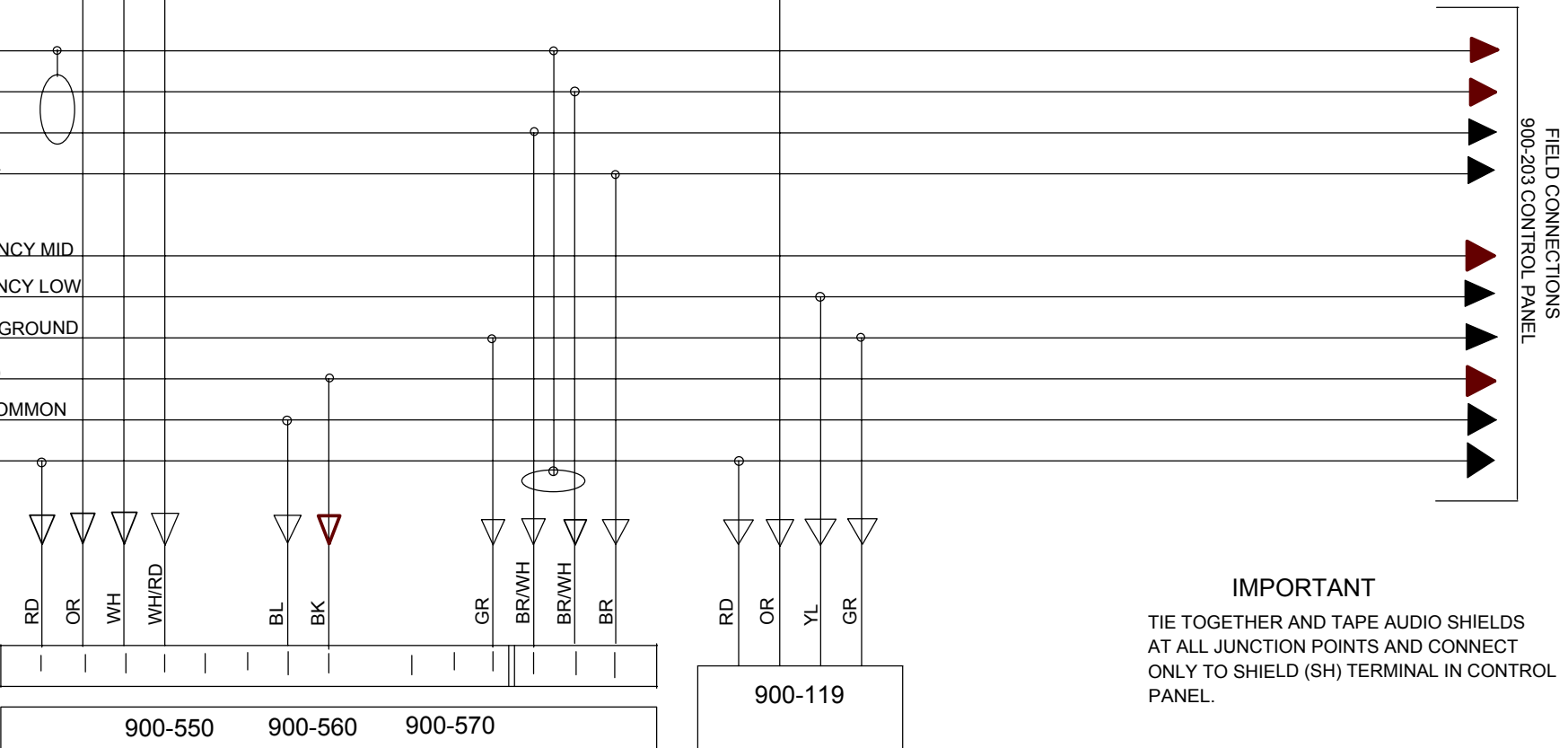

Engineered Electronics, Inc
 4301 Industrial Access Road
 Douglasville, Ga. 30134

TITLE
AUDIO VISUAL PATIENT STATION WITH ASSOCIATED EMERGENCY STATION

SIZE A	DRAWING NO. 102-009	REV
SCALE	DATE 4-11-02	

IMPORTANT

TIE TOGETHER AND TAPE AUDIO SHIELDS AT ALL JUNCTION POINTS AND CONNECT ONLY TO SHIELD (SH) TERMINAL IN CONTROL PANEL.