



**ENGINEERED
ELECTRONICS, INC.**

**GUARDIAN
PATIENT PROTECTOR 900-366
AND 900-367 SEAT SWITCH**

FEATURES

- * EASILY MOUNTED TO BACK OF WHEELCHAIR WITH SNAP CLIP
- * NO CONNECTION TO PATIENT OR CLOTHING
- * NO COMPLICATED ADJUSTMENT
- * LOCAL AUDIBLE ALARM WHEN ACTIVATED
- * PRESSURE ACTIVATED ALARM SEAT SWITCH
- * SEALED AND WATER PROOF SEAT SWITCH
- * SUPERVISED ALARM IF SEAT SWITCH IS DISCONNECTED
- * OPERATES ON ONE 9 VOLT BATTERY
- * LOW BATTERY INDICATION



DESCRIPTION

The 900-366 Guardian Patient Protector with 900-367 seat switch is designed for institution or home to alert caregiver if patient attempts to exit wheelchair. Simply place the water proof pressure switch in wheel chair bottom, insert the 1/4 inch plug into sensor position on patient protector and the system is ready to operate.

The Guardian Patient Protector can also be used in conjunction with an existing nurse call system by plugging in optional 900-808 adapter cable to room station.

ACCESSORIES

900-808 SYSTEM CORD

PARTS

100-993 CHAIR SNAP CLIP
100-437 BATTERY



ENGINEERED ELECTRONICS, INC. * 4301 INDUSTRIAL ACCESS ROAD * DOUGLASVILLE, GA. 30134

GENERAL OPERATION

Removal of pressure from the seat switch or disconnecting seat switch from patient protector will activate sounder until reset by caregiver.

BATTERY REPLACEMENT

Replace battery immediately if sounder chirps every 45 seconds when in the alarm position. To replace battery remove four screws on back of Patient Protector and replace with standard 9V battery.

OPERATION FOR MOVEMENT MANAGEMENT

1. Attach Guardian Patient Protector to back of wheelchair with snap clip.
2. Place seat switch in bottom of wheelchair with cable toward back of chair. Position switch toward middle of chair so majority of patients weight will be toward center of switch. Place cushion or Incontinence pad over switch if desired.
3. Connect seat switch to patient protector by inserting 1/4 inch phone plug into jack marked sensor.
4. With patient in chair press activation switch located on side of patient protector to the on position. A double beep should be heard to acknowledge the unit is set in alarm condition.
5. When the patients weight begins to shift off the chair, the alarm will be activated and will continue until reset by caregiver.

OPERATION FOR SYSTEM ALARM

1. Plug optional 900-808 system cord into jack marked "SYSTEM".
2. Plug other end of 900-808 cord into existing nurse call room station.

